***magnification***

***ClearView GO***

***User manual***  
***Gebruikershandleiding***  
***Manuel d'utilisation***

***Manuale utente***  
***Manual del usuario***  
***Bedienungsanleitung***

***Optelec***

## Contents

<b>Configuration</b> .....	<b>10</b>
<b>1. Introduction</b> .....	<b>12</b>
1.1. About this manual .....	12
1.2. Instructions before use .....	12
1.3. What's in the box .....	12
<b>2. Getting started</b> .....	<b>13</b>
2.1. Opening the ClearView GO .....	13
2.2. Closing the ClearView GO .....	14
2.3. Camera positions .....	15
<b>3. Buttons</b> .....	<b>16</b>
<b>4. Hot keys &amp; features</b> .....	<b>18</b>
4.1. Line & Mask .....	18
4.2. Battery status .....	19
4.3. HDMI OUT mode .....	19
4.4. Scroll the frozen image .....	20
4.5. Power saving mode .....	20
4.6. Color mode selection .....	20
4.7. Self-view & Distance-view mode .....	21
<b>5. Menu</b> .....	<b>22</b>
5.1. Navigating the Menu .....	22
5.2. Menu options .....	22
Color Mode .....	22
Zoom ratio.....	23
LED brightness .....	23
Reset.....	23
5.3 More Settings .....	24
Language .....	24
Capture Scroll Speed .....	24
Key Lock Mode .....	24
Personal & Public .....	24
Power Saving Mode .....	24
<b>Technical Specifications</b> .....	<b>25</b>
ClearView GO 15" .....	25
ClearView GO 17" .....	26
<b>Safety, Care &amp; Regulatory Notices</b> .....	<b>27</b>

## Inhoud

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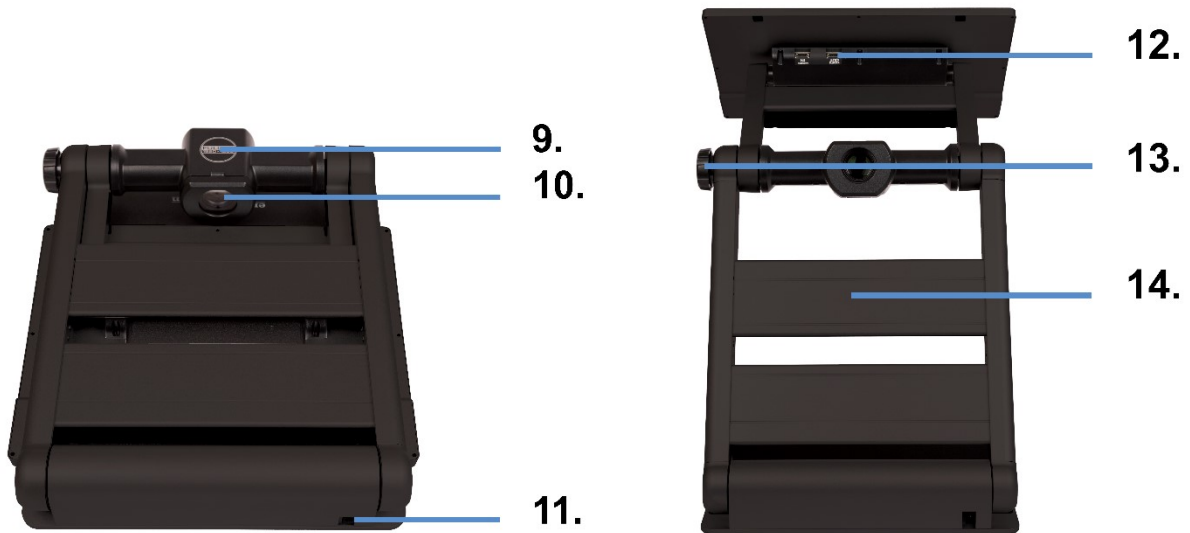
# Configuration

1. Power on/off
2. Image optimizer wheel (adjusts brightness and contrast)
3. Snapshot button
4. Zoom wheel
5. Color Mode button
6. Natural Color button
7. Super Function Key
8. LCD screen
9. Camera
10. Close up lens (open this lens for distance view)
11. DC jack (insert the power adapter here to charge the battery)
12. HDMI output
13. Camera Positioning wheel
14. LED lights

**Note:** With the camera ⑨, be careful not to allow foreign material to adhere to the camera lens. If any foreign material is on the lens, wipe it off with the provided microfiber cloth.

# 1. Introduction

Congratulations on choosing the ClearView GO. ClearView GO offers the same reading comfort, ease of use, and functionality as a desktop video magnifier, but in a compact and foldable design. It is the ideal solution for people who need magnification at different locations. With ClearView GO, you can comfortably read text and view objects, at home, office, classroom, or on the go. Its foldable design makes it extremely easy to take the ClearView GO with you or to store it when you do not need it.

## 1.1. About this manual

This manual will help you to become familiar with the ClearView GO features and operation. Please read this manual thoroughly before using your ClearView GO. If you have any questions or suggestions concerning the use of this product, please contact your Optelec distributor or Optelec's headquarters by consulting the contact information page located at the end of this manual. We greatly appreciate your feedback. We hope you enjoy working with your ClearView GO.

## 1.2. Instructions before use

The battery must be fully charged prior to the initial use. Connect the supplied power adapter to the DC input jack to charge the battery. While charging, a red light is illuminated. When fully charged, the light is green.

Remove the LCD protection film before using the unit. Be careful not to allow foreign material to adhere to the camera lens. If any foreign material is on the lens, wipe it off with the provided microfiber cloth.

## 1.3. What's in the box

The ClearView GO packaging contains the following items. If any of these items are missing from your package, please contact your distributor.

1. ClearView GO portable magnifier
2. Power adapter
3. Carrying pouch (15" model only)
4. User manual
5. Microfiber cloth for cleaning the display

## 2. Getting started

### 2.1. Opening the ClearView GO

1.

2.

3.

4.

5.

6.

1. Place the folded ClearView GO on a stable surface such as a table.
2. Press on the base plate with one hand, while the other hand lifts the top of the body frame.
3. Lift the arm until it locks in place.
4. Put both hands on the left and right sides of the LCD monitor.
5. Pull the LCD monitor towards you.
6. Push the LCD monitor up using both hands and adjust it to a suitable position.

## 2.2. Closing the ClearView GO

1.

2.

3.

4.

5.

1. Put both hands on the left and right sides of the LCD monitor.
2. To close the ClearView GO, lower the LCD monitor using both hands.
3. Push the LCD monitor toward the main body frame.
4. Place both hands on top of the body frame.
5. Lower the top of the body frame until it is closed.

## 2.3. Camera positions

1.

2.

3.

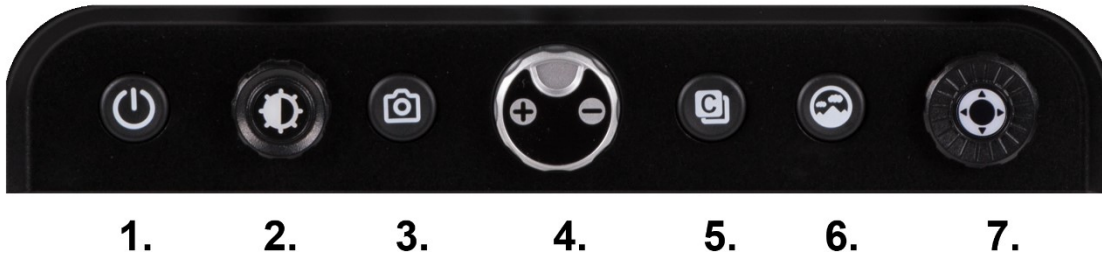
You can switch between modes by rotating/adjusting the Camera Positioning wheel.

1. Reading mode

2. Distance-view mode (open the close-up lens for distance viewing)

3. Self-view mode (mirror mode)

## 3. Buttons

### 1. Power On/Off button

To power the device on/off, press the button for two seconds.

### 2. Image optimizer wheel

Provides different functions:

- When in Natural color mode, turn the knob to adjust the brightness of the display (five levels).
- When in a high contrast color mode, turn the knob to adjust the image contrast. Press it to return to the default setting.
- To toggle auto focus on/off, press and hold the button for two seconds.

### 3. Snapshot button

Briefly press the Snapshot button ③ to freeze the image.

- You can zoom in/out on the frozen image and adjust color and screen brightness.
- When the image is magnified, you can pan around the image using the Super Function Key ⑦.

Press the Snapshot button again to return to live mode.

### 4. Zoom wheel

To adjust the magnification, turn the knob. The level of magnification depends on the size of ClearView GO that you own.

- ClearView GO 15":  
Magnification: 1.4x to 20x, 2.0x to 32x (default), or 3.5x to 60x.  
Additional magnification in captured image: 5.5x.

- ClearView GO 17":  
Magnification: 1.4x to 25x, 2.3x to 35x (default), or 3.5x to 70x.  
Additional magnification in captured image: 5.5x.

Different magnification levels can be set in the Menu. Press and hold the Zoom wheel to access the Menu and select Zoom Ratio to adjust. For more information on how to do this, refer to the "[Zoom ratio](#)" section in chapter 5.2.

## 5. Color mode button

The image on the screen can be viewed in eight color modes: the natural color mode and seven high contrast colors:

- Natural color mode
- White text on black background
- Black text on white background
- Yellow text on black background
- Black text on yellow background
- Green text on black background
- Blue text on yellow background
- Yellow text on blue background

Up to 35 color modes can be set using the menu.

## 6. Natural Color button

Press the Natural Color button (⑥) to view pictures or documents in natural color.

Press and hold the Natural Color button (⑥) to use the Find function when magnified. The Find function allows you to quickly zoom out and see the entire image. Releasing the button returns the camera to the original magnification level.

## 7. Super Function Key

Used for panning/scrolling the frozen image when magnified. Refer to [chapter 4.4](#).

Used to select and apply the line/mask function. Refer to [chapter 4.1](#).

# 4. Hot keys & features

## 4.1. Line & Mask

Press and hold the Super Function Key (7) to activate or deactivate Line & Mask. When Line & Mask is active, the direction key and wheel of the Super Function Key (7) work as follows.

- Left and right ◀ ▶ : Choose the function.

1. Horizontal line	
2. Vertical line	
3. Horizontal mask	
4. Vertical mask	

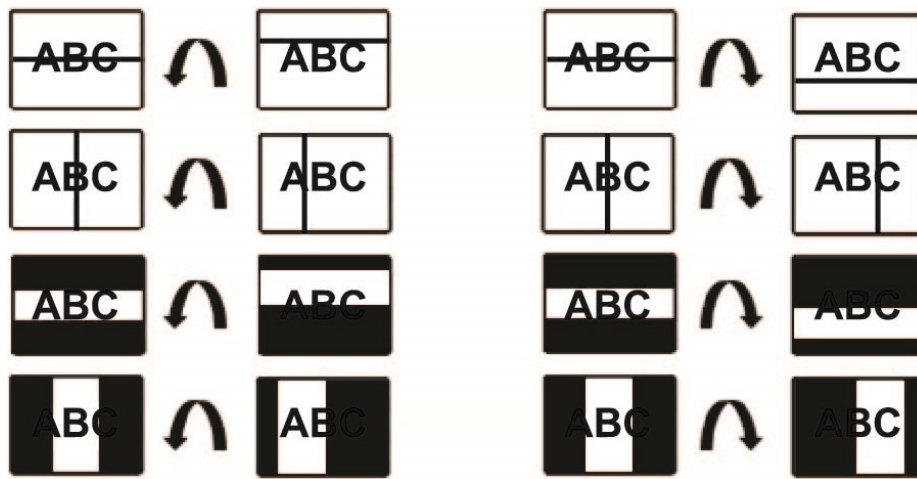
- Up and down ▲ ▼ : Increase or decrease the thickness of a line and the distance between masks.

	▲			▲	
	▼			▼	
	▲			▲	
	▼			▼	

- Turning the wheel moves the position of the line or mask.

1) Counterclockwise

2) Clockwise

## 4.2. Battery status

The status of the remaining battery power is not continuously displayed. The battery charging icon is displayed when the power adapter is connected.

To check the status of the remaining battery, press the Snapshot button (3) for three seconds or more.

## 4.3. HDMI OUT mode

When the HDMI output cable is connected, the LCD display is blank and the image is displayed on an external monitor. Switch between the LCD display and an external monitor by pressing the Super Function Key (7).

## 4.4. Scroll the frozen image

Adjust the zoom level as desired to frame the entire area you would like to read. Then, press the Snapshot button (3) to freeze the image.

When you press the Zoom wheel (4) while in the frozen image, a horizontal line marker appears. You can move this line by rotating the Super Function Key (7).

You can zoom in/out on the frozen image with the Zoom wheel (4). When the image is magnified, you can pan around the frozen image using the up/down and left/right direction keys around the Super Function Key (7). When you pan to the end of a frozen image, you can advance to the next paragraph or line by releasing the left or right direction key and pressing it again.

At any time while panning the image, your current reading position is displayed by pressing and holding the Natural Color button (6). The location of the magnified area is shown within the entire field of view.

## 4.5. Power saving mode

If you leave the unit turned on and unused for 10 minutes, the LCD screen turns off to conserve battery power. If the unit is left for three additional minutes, the power is automatically switched off. For more information about how to adjust these times, refer to the “Power saving mode” section in [chapter 5.3](#).

## 4.6. Color mode selection

Press and hold the Color Mode button (5) and rotate the Zoom wheel (4) to cycle through the available color modes. For more information about how to select colors, refer to the “Color Mode” section in [chapter 5.2](#).

- Color Mode button (5) + Zoom wheel (4), Right: Next color mode
- Color Mode button (5) + Zoom wheel (4), Left: Previous color mode
- Press the Color Mode button repeatedly to cycle through the available color modes.

## 4.7. Self-view & Distance-view mode

You can change the viewing mode by moving the camera with the Camera Positioning wheel (13).

### Self-view mode (Mirror mode)

Use when the direction of the camera is pointing towards you.

- The camera switches automatically into self-view mode.
- The magnification range is minimized.
- The camera switches automatically to natural color mode.

### Distance-view mode

Use this mode to view a distant object, such as a whiteboard. Open the close-up lens when using this mode.

- The camera switches automatically into distance-view mode.
- The magnification ratio is minimized.
- The camera switches automatically to natural color mode.

## 5. Menu

You use the Menu to customize the product. This section describes how to navigate the Menu and select options to change the look and feel of your ClearView GO.

### 5.1. Navigating the Menu

Press and hold the center of the Zoom wheel (4) to enter and exit the Menu.

Once in the Menu, you can change or save settings by using the Zoom wheel (4) or Super Function Key (7).

- Move around the menu by turning the Zoom wheel (4) or the Super Function Key (7).
- Select an option by pressing the center of the Zoom wheel (4) or Super Function Key (7).
- To return to the previous screen, press and hold the center of the Zoom wheel (4) or Super Function Key (7).

### 5.2. Menu options

The following menu options are available: **Color Mode**, **Zoom Ratio**, **LED Brightness**, **More Settings**, and **Reset**.

Within the **More Settings** menu, the following options are available: **Language**, **Capture Scroll Speed**, **Key Lock Mode**, **Personal & Public**, and **Power Saving Mode**.

#### Color Mode

Using the **Color Mode** option, you can add or remove your preferred high contrast color combinations. The default is a selection of seven high contrast color modes.

- Default: Seven high contrast color modes selected.
- Extend: All 35 color modes selected.

Select your preferred color modes using the Zoom Wheel ④ or the Super Function Key ⑦. Enable and disable a color mode by pressing the Zoom Wheel ④. A color mode is enabled when circled in red. A color mode is disabled when circled in gray. Press **save** to apply the changes you made.

### **Zoom ratio**

For ClearView GO 15", the default magnification range is 2.0x to 32x. You can set the magnification range to:

- 1.4x to 20x
- 2.0x to 32x
- 3.5x to 60x

For ClearView GO 17", the default magnification range is 2.3x to 35x. You can set the magnification range to:

- 1.4x to 25x
- 2.3x to 35x
- 3.5x to 70x

### **LED brightness**

With this option, you can adjust the LED brightness through 11 different levels, including turning the lights completely off.

### **Reset**

This menu resets all settings back to default values.

## 5.3 More Settings

If you select **More Settings**, you can adjust the following functions: **Language**, **Capture Scroll Speed**, **Key Lock Mode**, **Personal & Public**, and **Power Saving Mode**.

To save your updated settings, select **Save & Exit** or press and hold the Image Optimizer wheel (2).

### Language

This menu enables you to select your preferred language.

### Capture Scroll Speed

When this option is selected, you can adjust the capture scroll/panning speed. Or, use the Image Optimizer wheel (2). You can test the scroll speed in real-time using the Super Function Key (7). Make sure that you are zoomed in to be able to test the scroll speed.

### Key Lock Mode

If you select **Yes**, only three buttons are available: Zoom wheel (4), Color Mode button (5), and Natural Color button (6). If you select **No**, all buttons are available.

### Personal & Public

With **Personal** mode, all current customized user settings are remembered after the device is rebooted. With **Public** mode, all settings are returned to their factory default values after the device is rebooted. Public mode is recommended when a device is used by several people, such as in a library.

### Power Saving Mode

To conserve battery life, you can adjust the length of time before the LCD automatically turns off after a period of non-use. You can also adjust the time when all power is turned off. The defaults settings are:

- **Auto LCD OFF** time is 10 minutes.
- **Auto Power OFF** time is three minutes.

# Technical Specifications

## ClearView GO 15”

<b>LCD screen size</b>	15.6 inches (396mm), 16:9 wide screen
<b>LCD resolution</b>	1920 x 1080 (full HD)
<b>LCD rotation angle</b>	180 degrees
<b>Camera specifications</b>	1920 x 1080, 60 fps Auto focus
<b>Dimensions</b>	Folded: approx. 15.1 x 16 x 2.5 inches or approx. 384 x 405 x 64 mm In use: approx. 15.1 x 15.75 x 22.7 inches or approx. 384 x 400 x 577 mm (max. height)
<b>Light weight</b>	10.65 lbs / 4.8 kg
<b>Color modes</b>	Default 8, including natural color (extended 35)
<b>Magnification range</b>	Live image mode: 1.4x to 20x, 2.0x to 32x (default), or 3.5x to 60x. Additional magnification in captured image: 5.5x
<b>Camera mode</b>	Reading, Distance-view, Self-view
<b>Battery capacity</b>	16.8V, 5200mAh
<b>Battery weight</b>	0.90 lbs / 400 g
<b>Power consumption</b>	14~20W (in operation)
<b>Battery life</b>	5 hours
<b>Input power rating for Adapter</b>	100-240V, 50-60Hz, 1.7A
<b>Output power rating for unit</b>	DC +19V-3.42A
<b>Connectivity</b>	HDMI OUT

**ClearView GO 17”**

<b>LCD screen size</b>	17.3 inches (439mm), 16:9 wide screen
<b>LCD resolution</b>	1920 x 1080 (Full HD)
<b>LCD rotation angle</b>	180 degrees
<b>Camera specifications</b>	1920 x 1080, 60 fps Auto focus
<b>Dimensions</b>	Folded: approx. 16.2 x 17 x 2.5 inches or approx. 411 x 432 x 64 mm In use: approx. 16.2 x 17.5 x 25.7 inches (max. height) approx. 411 x 445 x 653 mm (max. height)
<b>Light weight</b>	12.05 lbs / 5.465 kg
<b>Color modes</b>	Default 8, including natural color (extended 35)
<b>Magnification range</b>	Live image mode: 1.4x to 25x, 2.3x to 35x (default), or 3.5x to 70x. Additional magnification in captured image: 5.5x
<b>Camera mode</b>	Reading, Distance-view, Self-view
<b>Battery capacity</b>	16.8V, 5200mAh
<b>Battery weight</b>	0.90 lbs / 400 g
<b>Power consumption</b>	14~20W (in operation)
<b>Battery life</b>	5 hours
<b>Input power rating for Adapter</b>	100-240V, 50-60Hz, 1.7A
<b>Output power rating for unit</b>	DC +19V-3.42A
<b>Connectivity</b>	HDMI OUT

# Safety, Care & Regulatory Notices

- Negligent use or misuse of the product may cause a fire, personal injury or serious damage.
- Do not use damaged power plug or unsuitable outlet to prevent electric shock or fire.
- Do not touch power plug with wet hands to prevent electric shock.
- Misuse of the product may result in personal injury or damage to the unit.
- Use only the adapter supplied with the product. Other adapters may damage the unit.
- Do not use this unit in moist environments like a bathroom or shower room.
- Do not place heavy objects on the unit.
- Do not attempt to disassemble, remodel, or repair the unit. It may damage the unit and will void the warranty.
- After six (6) months usage, the lifespan of the battery may be shortened.
- Keep the product out of direct sunlight and extreme temperatures.
- The display may not work properly.
- The recommended operating temperature is between 0 ~ 35 °C (32 ~ 95 °F)

## Precautions for use

- Follow all warnings, cautions, and instructions marked on the product.
- Use ClearView GO on a stable, level surface such as a solid desk or table, according to its intended design specifications.
- Avoid placing ClearView GO on an unstable table, cart, or stand. It may fall, causing serious damage to the device or bodily injury.
- Use ClearView GO indoors only—it should NOT be exposed to extreme or rapid changes in temperature or humidity. Do not place ClearView GO near a radiator, heater, or areas exposed to long hours of direct sunlight. These conditions may cause damage to the device's electronics or optics.
- At least once per week, clean ClearView GO with a soft dry cloth or an electrostatic dusting cloth. DO NOT use liquid cleaners on plastic surfaces. No additional methods of sterilization or cleaning are recommended or required.
- Do not use ClearView GO near flammable materials or liquids.

## Safety and care

To ensure your safety and obtain the most benefit from ClearView GO, review these safety precautions carefully.

ClearView GO enlarges and enhances text and objects in order to improve the quality of life for people who are visually impaired. The extent to which this product can improve the ability to see images depends on a person's individual condition and level of visual impairment.

ClearView GO is not designed to restore vision, correct eye conditions, or halt degenerative vision loss. An eye care professional can determine the usefulness and suitability of ClearView GO for a patient's individual condition.

## Caring for ClearView GO

To care for your ClearView GO, follow these recommendations:

- When ClearView GO is not in use and not connected to external power, always turn the power off by pressing the Power button for two seconds.
- Ensure that your ClearView GO is completely off before cleaning it.
- Clean ClearView GO with a soft dry cloth or an electrostatic dusting cloth. DO NOT use liquid cleaners on its plastic surfaces.
- To prolong the life of ClearView GO, remove the battery from the device if it will not be in use for one week or more.
- The protective glass covering the camera can be cleaned as needed with the lens cloth provided within the packaging of ClearView GO. DO NOT use liquid cleaners on the plastic surfaces. No additional methods of sterilization or cleaning are recommended or required.

## Transporting ClearView GO

Carry the device by firmly grasping the ClearView GO main body when it is in the folded position.

**Warning:** Do not carry ClearView GO by the camera head since this may result in damage to your device.

The ClearView GO 15" comes with a convenient carrying case for transport. To prevent the device from being accidentally dropped and damaged, always return the ClearView GO to the carrying case when not in use. Disconnect and secure all cables before moving the device.

## FCC Compliance Notice

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in some instances. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, you are encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment to a wall outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Unauthorized changes or modifications to this system can void your authority to operate this equipment.

## Industry Canada (IC) Notice

This Class B digital apparatus complies with Canadian ICES-003.

**This unit is an EMC-registered device.**

This device bears the CE and FCC, KC approval mark

### **Additional mechanical safety precautions**

Your ClearView GO has moving parts. Do not attempt to forcibly reposition or adjust the parts. It may cause damage to the device.

**Note:** Use caution when moving parts or when transporting the ClearView GO to avoid possible injury to hands and fingers.

### **Susceptibility to interference**

There may be temporary degradation of the image when ClearView GO is subjected to a strong radio frequency field, electrostatic discharge, or transient electrical noise. An electrostatic discharge (caused by static electricity) may cause the screen to go blank. If this happens, follow these steps:

1. Disconnect the DC adapter power supply from the electrical outlet.
2. Wait 30 seconds.
3. Connect the DC adapter power supply again.

### **Additional electrical safety precautions**

Read these safety precautions before using ClearView GO:

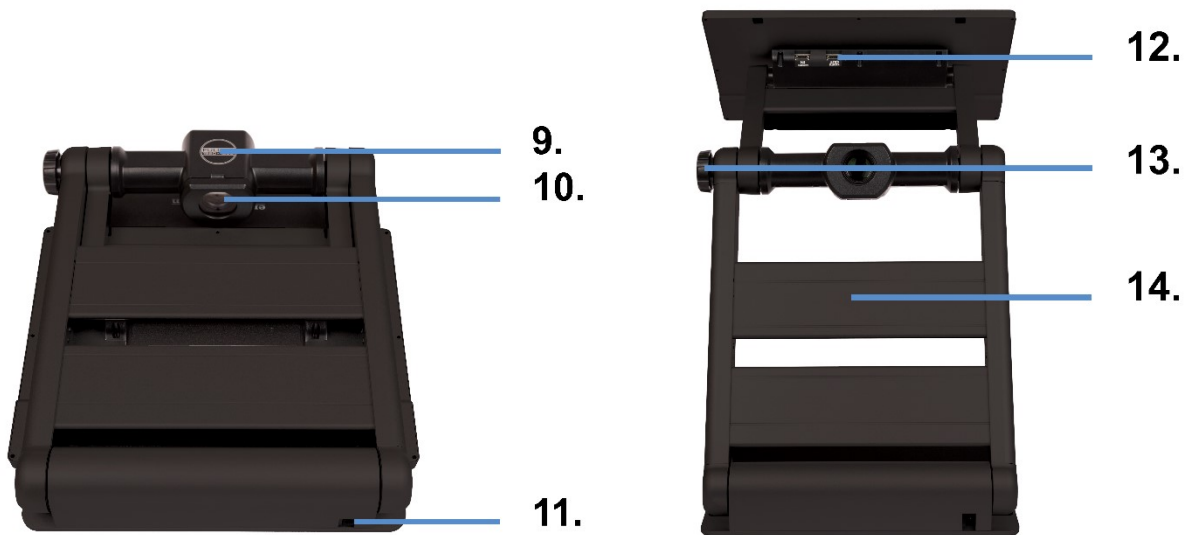
- Using any power supply other than the one provided by Optelec for ClearView GO will **void the warranty**.
- If using DC power, the ClearView GO device should be near a wall outlet that is easily accessible.
- Do not use ClearView GO in, near, or around water. ClearView GO is not designed to offer protection against contact with any liquids.
- Use only the power supply provided with your ClearView GO. Using unauthorized power supplies may cause serious damage to ClearView GO's electronic components.
- Always place the power cord away from traffic areas to avoid damage to the power supply.
- Prevent the risk of electrical shock by NOT attempting to disassemble the device. Refer all service to an authorized repair facility.
- Ensure that you make all proper cable connections.

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Copyright 2020 Optelec, the Netherlands
All rights reserved
## **Optelec GmbH**