
Optelec ClearReader⁺

User Manual
Gebruikershandleiding
Manuel d'utilisation
Manuale utente
Manual del usuario
Bedienungsanleitung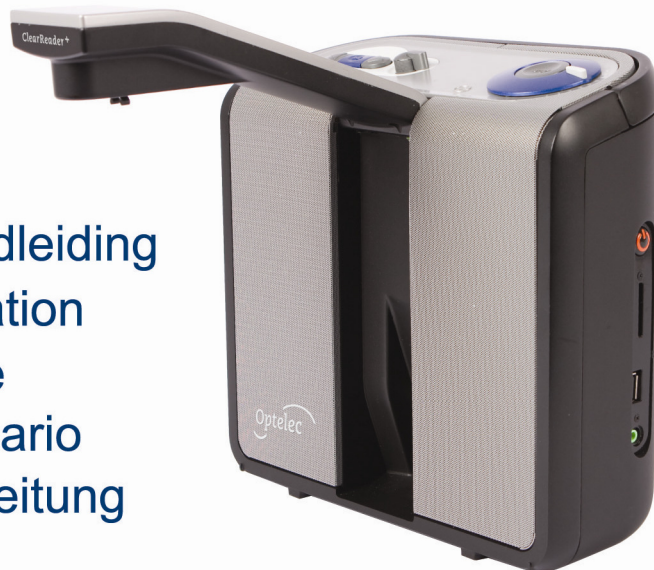

The ClearReader+ will be helpful for low vision and blind people in reading.

WEEE Notice

The directive on Waste Electrical and Electronic Equipment (WEEE), which entered into force as European law on 13th February 2003, resulted in a major change in the treatment of electrical equipment at end-of-life.

The purpose of this Directive is, as a first priority, the prevention of WEEE, and in addition, to promote the reuse, recycling and other forms of recovery of such wastes so as to reduce disposal.

The WEEE logo (shown at the left) on the product or its box indicates that this product must not be disposed of or dumped with your other household waste. You are liable to dispose of all your electronic or electrical waste equipment by relocating over to the specified collection point for recycling of such hazardous waste. Isolated collection and proper recovery of your electronic and electrical waste equipment at the time of disposal will allow us to help conserve natural resources. Moreover, proper recycling of the electronic and electrical waste equipment will ensure safety of human health and the environment. For more information about electronic and electrical waste equipment disposal, recovery, and collection points, please contact your local city centre, household waste disposal service, shop from where you purchased the equipment, or manufacturer of the equipment.

RoHS Compliance

This product is in compliance with Directive 2002/95/EC of the European Parliament and of the Council of 27 January 2003, on the restriction of the use of certain hazardous substances in electrical and electronic equipment (RoHS) and its amendments.

European Union Notice

Products with CE marking comply with the R&TTE Directive (99/5/EC), the EMC Directive (89/336/EEC), and the Low Voltage Directive (73/23/EEC) issued by the Commission of the European Community.

Compliance with these directives implies conformity to the following international standards:

IEC 60950-1 2004;

IEC 55022

IEC 55024 (IEC61000-4-2 excluded)

FCC

FCC part 15 Class B

ClearReader⁺

User Manual

OPN: CR-BA-EU/US/AU Version 1.2

© 2010 Optelec, the Netherlands

All rights reserved

Optelec

P.O. Box 399

2990 AJ Barendrecht

The Netherlands

Phone: +31 (0)88 678 34 44

E-mail: info@optelec.nl

Internet: www.optelec.com

Contents

1. Introduction.....	6
About this manual	6
What's in the box.....	7
2. Getting to Know your ClearReader⁺	8
The Optelec ClearReader ⁺ layout.....	8
Buttons	10
Connections	11
3. Getting Started.....	12
Using the ClearReader ⁺ for the first time	12
Opening the camera arm.....	12
Positioning a document.....	12
Turning on the ClearReader ⁺	13
Standby mode	13
Turning off the ClearReader ⁺	14
Reading text.....	14
Reading tables and spreadsheets.....	14
Adjusting the volume	14
Adjusting the reading speed.....	15

Play or pause reading	15
Navigate back in the text.....	15
Navigate forward in the text.....	15
Selecting the reading voice or language	16
4. Reading Languages and Voices of the ClearReader⁺	17
5. Power and Battery Use.....	18
Checking the battery status.....	18
Charging the ClearReader ⁺ battery	18
Extending the battery life.....	18
6. Troubleshooting	19
Appendix A: Safety	21
Appendix B: Technical information.....	22
Contact information – Optelec Offices Worldwide.....	123

1. Introduction

Congratulations on choosing the ClearReader⁺, an intuitive portable reading assistant developed by Optelec.

The Optelec ClearReader⁺ combines ease of use with high quality multilingual speech output and fast text recognition. It can capture any printed material such as letters, newspapers and books, and reads text aloud using a naturally sounding voice. The built-in speakers provide high-quality stereo sound. Thanks to the integrated battery you can enjoy reading without the need to be in close proximity of a mains power outlet.

If you have any questions or suggestions concerning the use of the Optelec ClearReader⁺, please contact your Optelec distributor using the contact information on the last page of this manual. Your feedback is greatly appreciated and we hope you enjoy using your ClearReader⁺.

About this manual

This manual will familiarize you with the features and easy operation of the ClearReader⁺. Please read this manual carefully before using your Optelec ClearReader⁺.

What's in the box

The ClearReader⁺ packaging contains the following:


- The Optelec ClearReader⁺
- A power supply
- A power cord
- This user manual

If any of these items are missing from your package, please contact your Optelec distributor.

2. Getting to Know your ClearReader+

This chapter will familiarize you with your Optelec ClearReader+

The Optelec ClearReader+ layout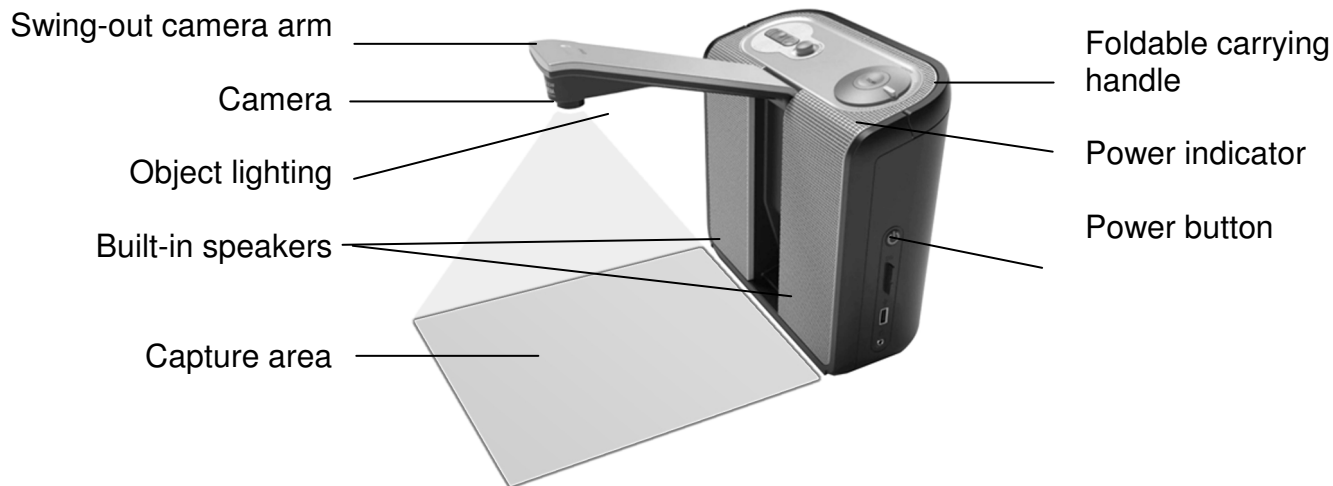


Swing-out camera arm	Open the arm to capture a document.
Camera	To take a snapshot, position the document in front of the ClearReader ⁺ . Use the document positioning indicators on the bottom left and right of the ClearReader ⁺ to position the document. The document area is approximately the width of the ClearReader ⁺ and the length of a regular A4 or letter size page.
Object lighting	The ClearReader ⁺ automatically provides sufficient lighting when taking a snapshot.
Built-in stereo speaker	Listen to your letters, newspaper or books.
Foldable carrying handle	Open the handle to lift and carry the ClearReader ⁺ comfortably.
Power indicator	Light is on when the power is on, blinks in standby mode, and is off when the power is off.
Power button	Press the round button on the right-hand side to turn the ClearReader ⁺ on or off or to put it into standby mode.

Buttons

Play / Pause button

Speed dial

Volume dial

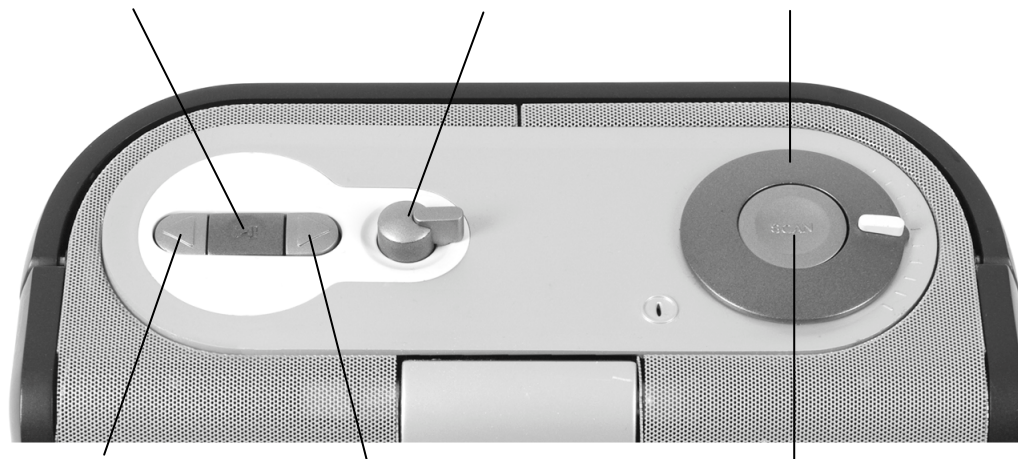Back button

Forward button

Scan button

Connections

Power connector

Connect the power adaptor to your ClearReader⁺.

Headphone out

Connect headphones if needed.

3. Getting Started

Using the ClearReader⁺ for the first time

When you use the Optelec ClearReader⁺ for the first time, the battery needs to be charged. To charge the battery connect the ClearReader⁺ to a mains power outlet using the ClearReader⁺ power supply included. Charging takes approximately 3 hours.

Opening the camera arm

To open the camera, locate the “push” sign on the camera arm at the front of the ClearReader⁺ between the speakers. Gently push on the camera arm until you hear a click. The arm will eject and move towards you. Close the camera arm in the same way by pushing the end of the arm back to the closed position until you hear a click.

Positioning a document

The Optelec ClearReader⁺ camera can capture an area of an A4 or letter-size page measured from the front side of the ClearReader⁺. To position the document, place it in front of the ClearReader⁺ between the two document positioning indicators with the top edge of the document aligned towards the front of the unit. The orientation of the document will not influence the recognition of the text.

Turning on the ClearReader⁺

To turn on the ClearReader⁺, press the round Power button on the right-hand side. The start-up process will take approximately 40 seconds when the ClearReader⁺ is powered down completely. You will hear a series of beeps to indicate that the ClearReader⁺ is starting up followed by the start-up sound. Now the Power indicator is lit up and the ClearReader⁺ is ready to use. If you turn on the ClearReader⁺ from the standby mode the start-up process will take approximately 10 seconds. Therefore, it is recommended to switch to standby mode instead of turning off the unit completely.

Standby mode

A short press of the Power button will put the ClearReader⁺ into standby mode. The power down sound will be played, the Power indicator will start blinking and the ClearReader⁺ will go to standby mode. In standby mode only a small amount of electrical power is consumed. On batteries the standby time of the ClearReader⁺ is approximately 40 hours. After 40 hours the ClearReader⁺ will power down. Please note that switching to standby mode takes approximately 12 seconds and the ClearReader⁺ will not react to any button press during this time.

Turning off the ClearReader⁺

Press and hold the Power button for approximately 6 seconds to power down the ClearReader⁺ completely. The power down sound will be played and the Power indicator will turn off.

Reading text

Press the Scan button to capture and start reading a document positioned beneath the camera. After pressing the Scan button, you will hear the snapshot sound and within a few seconds the ClearReader⁺ will begin reading to you.

Reading tables and spreadsheets

To read tables, receipts and bank statements, press and hold the Scan button for more than 2 seconds until you hear a clicking sound. A long press of the Scan button will enable the Table Reading mode and will disable the automatic column recognition making it possible to read across columns. Text will be read from left to right ignoring column borders.

Adjusting the volume

Turn the Volume dial clockwise and release it to slightly increase the volume of the reading voice. Alternatively, turn the dial counterclockwise and release it to slightly decrease the volume. To rapidly adjust the volume turn and hold the dial to the end position. When releasing the dial it will always move back to its start position.

Adjusting the reading speed

Turn the Speed dial clockwise and release it to slightly increase the reading speed. Alternatively, turn the dial counterclockwise and release it to slightly decrease the reading speed. To rapidly adjust the reading speed turn and hold the dial at the end position. When releasing the dial it will always move back to its start position.

Play or pause reading

The ClearReader⁺ will automatically start reading after pressing the Scan button. To pause reading press the Play / Pause button. Press this button once again to continue reading.

Navigate back in the text

Press the Back button to navigate to the previous word. Press and hold the Back button for 2 seconds to navigate to the previous sentence.

Navigate forward in the text

Press the Forward button to navigate to the next word. Press and hold the Forward button for 2 seconds to navigate to the next sentence.

Selecting the reading voice or language

To select your preferred reading voice or reading language, press and hold the Back button while turning the Speed dial at the same time. The ClearReader⁺ will switch to the next voice and announce the name of the newly selected voice and the language. For optimal performance the reading language and language of the document need to match. If the language of the document does not match the selected reading language, the text recognition will be of a lower quality and the recognition will take longer to read.

4. Reading Languages and Voices of the ClearReader⁺

The ClearReader⁺ is able to read in four different voices supporting up to a maximum of four languages. Check with your Optelec distributor for a list of available reading voices – these must be selected at the time of purchase.

5. Power and Battery Use

Checking the battery status

Press the Back and Forward buttons together to check the battery status. The ClearReader⁺ will announce the status in percentage, where 100% means a fully charged battery. If the battery is low, please charge the battery with the dedicated power supply.

Charging the ClearReader⁺ battery

To charge the ClearReader⁺, connect the mains adaptor supplied in the box to the power connector located at the rear side of the unit. Plug the other side of the adaptor into a power outlet.

Charging the ClearReader⁺ takes approximately 3 hours. A fully charged ClearReader⁺ can operate for approximately 5 hours. Please note that, for safety reasons, only the adaptor supplied by Optelec should be used to charge the ClearReader⁺.

Extending the battery life

To ensure the battery pack maintains maximum capacity, disconnect the ClearReader⁺ from the power source at least once a month and operate it on battery power until the battery pack is empty.

6. Troubleshooting

If the adaptor does not power the ClearReader⁺.

- Check the connections. Make sure the power adaptor is firmly connected to the ClearReader⁺ and to the power outlet.
 - If the adaptor still does not power the unit, contact your Optelec distributor.
-

If the battery does not power the ClearReader⁺.

- The battery may be empty. Connect the power adaptor to charge the battery.
-

If the text you are reading does not make sense.

- Make sure your reading language is corresponding with the language of your document.
-

If the text recognition takes unusually long before starting to read your document.

- Make sure your reading language is corresponding with the language of your document.
-

If the ClearReader⁺ does not read the whole line of a document / column / table.

- The automatic column recognition may be on. Press and hold the Scan button for 2 seconds to disable the column recognition.
-

If the ClearReader⁺ does not respond to the Scan button.

- Open the camera arm and try it again.
 - If the Scan button still does not respond, check if the unit is powered.
-

If the ClearReader⁺ does not respond to any button.

- Make sure the battery is not empty. If the battery is empty, connect the power adaptor to charge the battery.

Appendix A: Safety

- Handle the ClearReader⁺ with care. Rough handling will damage internal components.
- Contact your Optelec distributor to service this equipment if necessary.
- **Do not open or remove any parts** of the ClearReader⁺ as it will void the warranty.
- Do not try to remove the battery.
- Do not expose the ClearReader⁺ to excessive heat or direct sunlight to avoid the risk of fire.
- Do not use the ClearReader⁺ near inadequately shielded medical devices.
- To avoid risk of electrical damage, keep your ClearReader⁺ away from fluids and chemicals.
- Always unplug the ClearReader⁺ before cleaning. Use a soft damp cloth to clean the exterior. Do not use cleaning agents or abrasive materials as they will damage your unit.

Use of the ClearReader⁺ other than described in this manual will exclude it from warranty conditions.

Appendix B: Technical information

Dimensions (W x H x D): 24 x 23 x 10 cm (9.4 x 9.1 x 3.9 inches)

– Depth, camera arm open: 27 cm (10.6 inches)

– Height, camera arm open: 25 cm (9.8 inches)

– Height, handle open: 27 cm (10.6 inches)

Weight: 2.46 kg (5.4 lbs)

Camera: High resolution camera

Earphone connection: 3.5 mm stereo jack

Battery type: Li-ion

Charging Time: 3 hours

Battery life: Approximately 5 hours continuous use

Operation conditions:

Relative humidity: <70%, no condensation

Temperature: 0 to 40 °C

Storage and transport conditions:

Relative humidity: <95%, no condensation

Temperature: -20 °C to 60 °C

This device bears the CE and FCC approval mark

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Optelec Limited
Optelec GmbH